


**LCD DISPLAY**




**PORTABLE LCD**

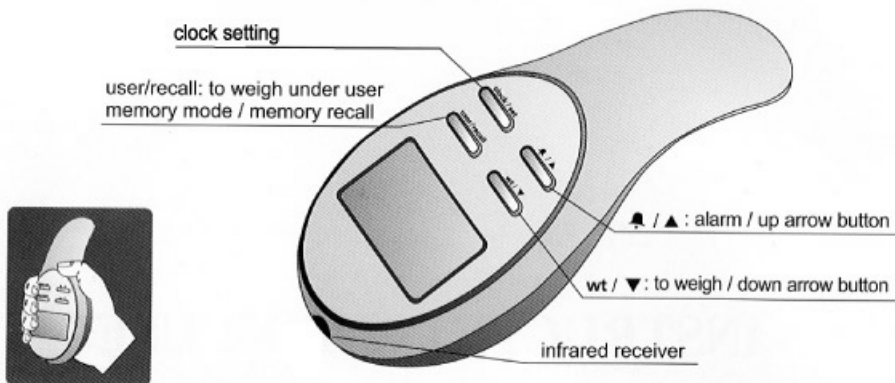


⚠ Attention: During the course of weighing, DO NOT cover the infrared receiver.

**CLOCK SETTING**

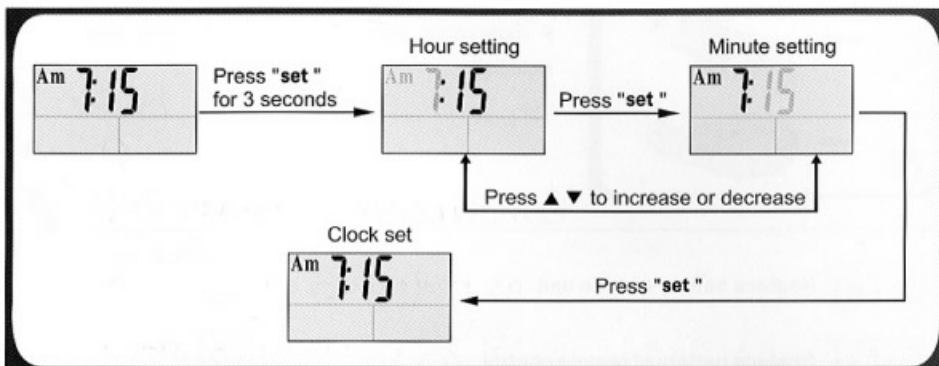


**ALARM SETTING**

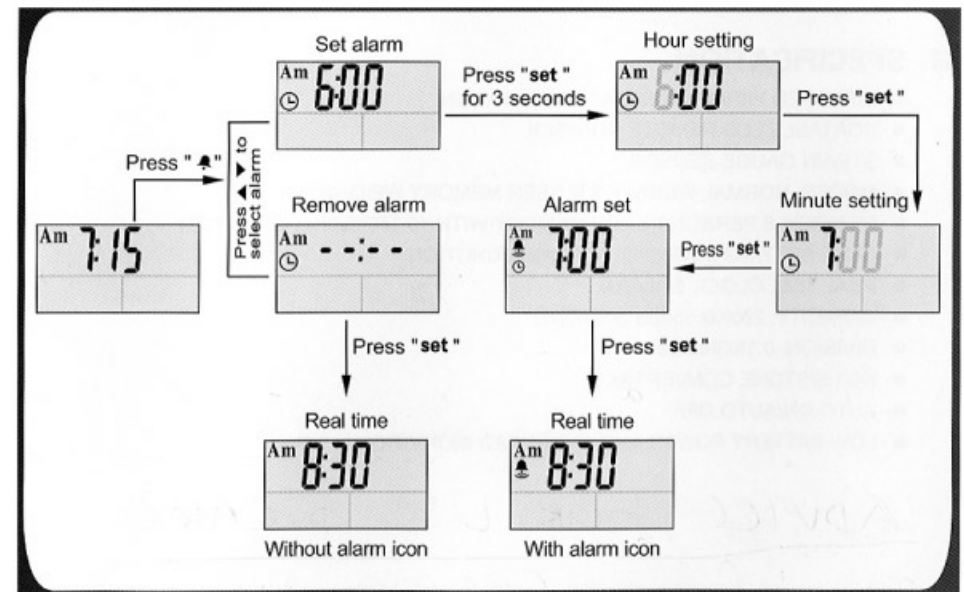


**TO WEIGHT UNDER NORMAL WEIGHING MODE**

▶ Step 1 : Select unit system

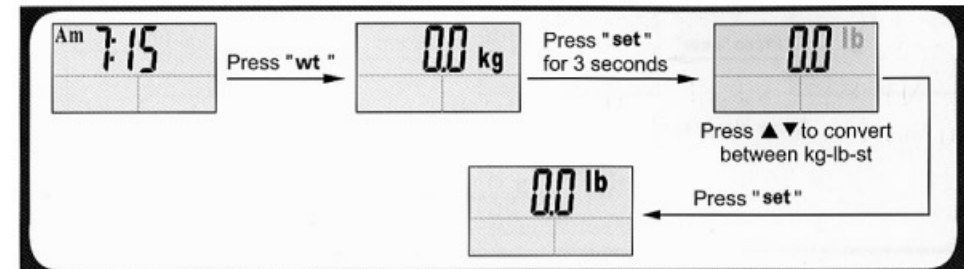


▶ Step 2 : To start weighing

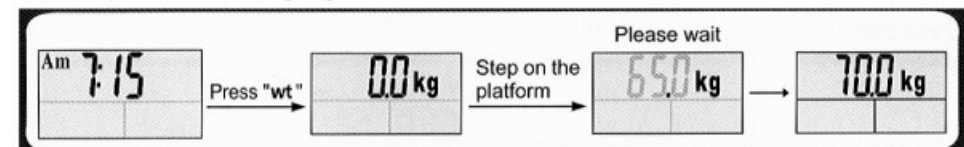


**TO WEIGH UNDER USER MEMORY WEIGHING MODE**

▶ Step 1 : Initial setting

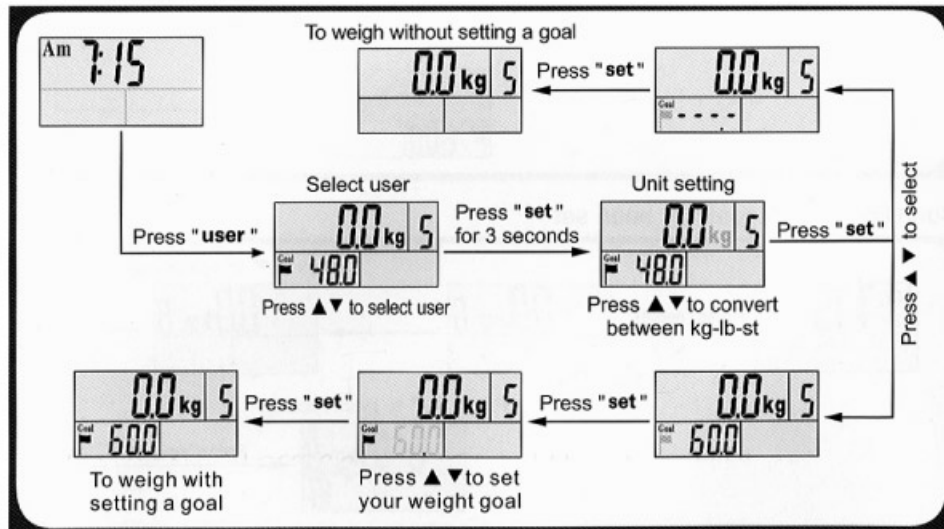

▶ Step 2 : To start weighing

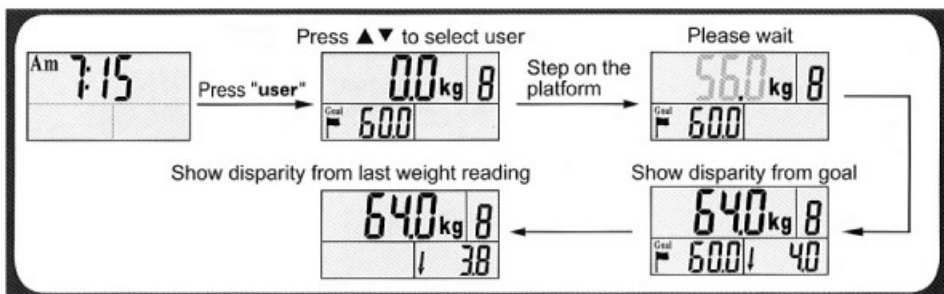


**SPECIFICATION**


- ▶ Infrared wireless data transmission
- ▶ Portable LCD remote control
- ▶ Strain gauge sensor
- ▶ Modes: Normal weighing & user memory weighing
- ▶ Memory: 8 persons (each memory with 10 tracking records)
- ▶ Goal setting & weight tracking function
- ▶ Clock & Alarm
- ▶ Capacity: 250kg, 550lb, 39st
- ▶ Division: 0.1kg/0.2lb
- ▶ KG/LB/STONE convertible
- ▶ Auto ON/OFF
- ▶ Low battery power & overload indication

**WEIGHT TRACKING**


▶ Condition 1: If a goal has been set

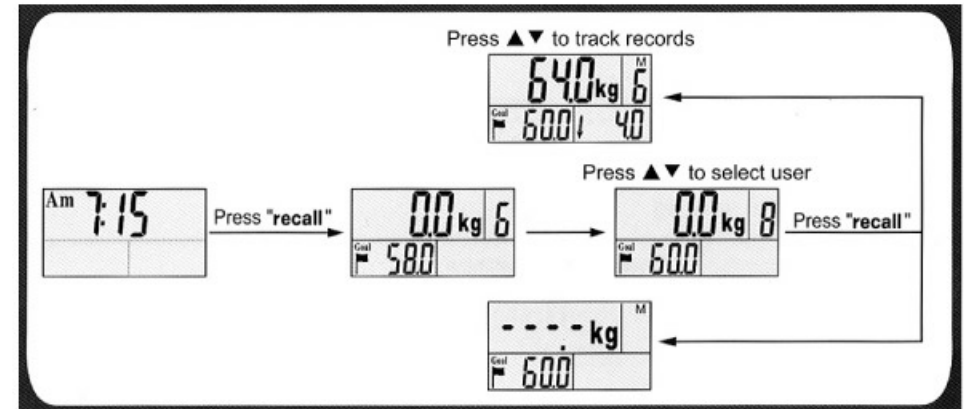

▶ Condition 2: If no goal has been set

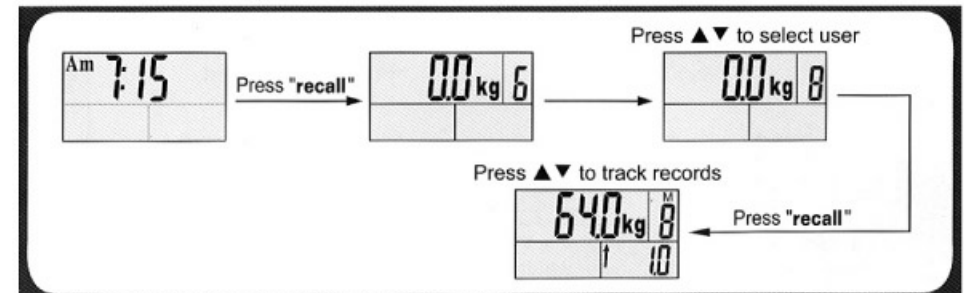

**TO REPLACE BATTERY**

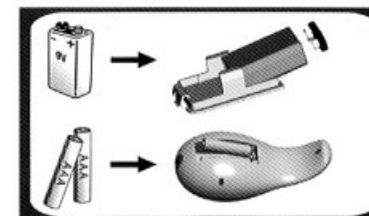

Main Unit : Takes 1x9v batteries (not included)  
 Remote control : Takes 2x1.5v AAA batteries (not included)

**b-Lo** Replace battery of main unit as batteries are low.

**b-Lo** Replace battery of remote control as batteries are low.

**Err** Main unit overload.